

NEW


LOCATOR F-Tx®
FIXED ATTACHMENT SYSTEM

SEAT AND REMOVE THE LOCATOR F-Tx® PROSTHESIS WITH ONE TOOL!

SEAT!

REMOVE!

THE LOCATOR F-Tx SEATING AND REMOVAL TOOL:

VERSATILE: One Tool for BOTH seating and
removal of the LOCATOR F-Tx prosthesis

PATIENT FRIENDLY: Quickly seat and
remove the prosthesis during patient
hygiene and maintenance visits


COST EFFECTIVE: Autoclavable for
multipatient use

**ZD ZEST DENTAL
SOLUTIONS®**

ZEST | DANVILLE MATERIALS | PERIOSCOPY

THE LOCATOR F-Tx® SEATING AND REMOVAL TOOL

QUICKLY SEAT AND REMOVE THE LOCATOR F-Tx PROSTHESIS


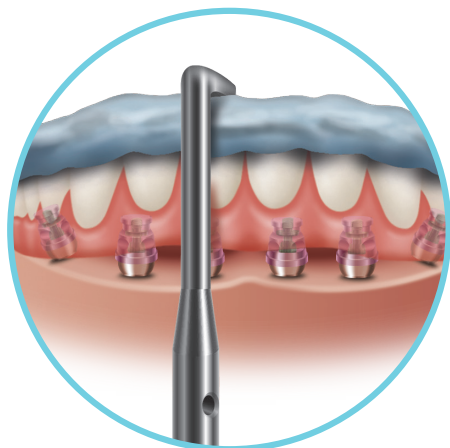


SIMPLE AND CONFIDENT SEATING

- The LOCATOR F-Tx Seating and Removal Tool has a specialized Seating Tip that allows for easy intraoral access and placement of the Tip over each Abutment location for quick seating.
- Simply activate the handle to deliver the necessary force to ensure Retention Balls are fully seated and the prosthesis is secure.

PATIENT FRIENDLY, QUICK REMOVAL

- The LOCATOR F-Tx Seating and Removal Tool saves valuable chair time during hygiene and maintenance visits increasing patient satisfaction and freeing up time for additional procedures.
- Simply loop the wire Removal Tip around the prosthesis and activate the handle to deliver the force required to disengage the Retention Balls from the Abutments.

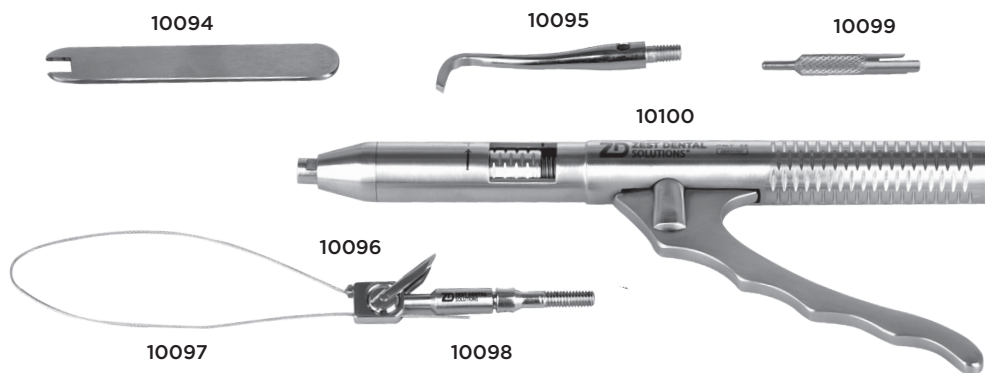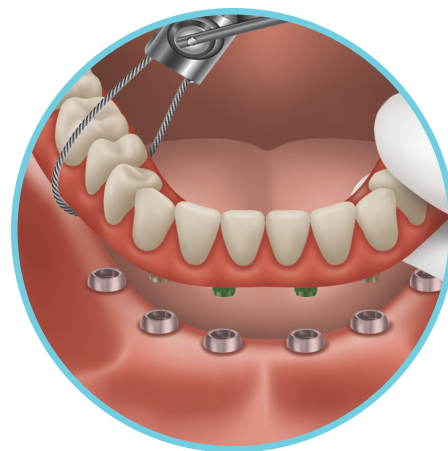

ITEM NUMBER	PRODUCT DESCRIPTION	PRICE
10100	LOCATOR F-Tx Seating and Removal Tool (includes Tool, Seating Tip, Removal Tip with Threaded Adapter, Wire/Lever Wrench and Tip Wrench).	\$239.00
10095	LOCATOR F-Tx Seating Tip	\$19.99
10096	LOCATOR F-Tx Removal Tip with Threaded Adapter	\$99.99
10097	LOCATOR F-Tx, Wire Replacement Part	\$19.99
10098	LOCATOR F-Tx, Threaded Adapter	\$19.99
10099	LOCATOR F-Tx Wire and Lever Wrench	\$19.99
10094	LOCATOR F-Tx Tip Wrench	\$12.99



ZEST | DANVILLE MATERIALS | PERIOSCOPY

ZEST DENTAL SOLUTIONS
2875 LOKER AVENUE EAST
CARLSBAD, CA 92010
USA

TEL: 800.262.2310
FAX: 760.743.7975
EMAIL: SALES@ZESTDENT.COM
WWW.ZESTDENT.COM